


Patient Station, Crest Replacement for Edwards System

7930C
7930LC

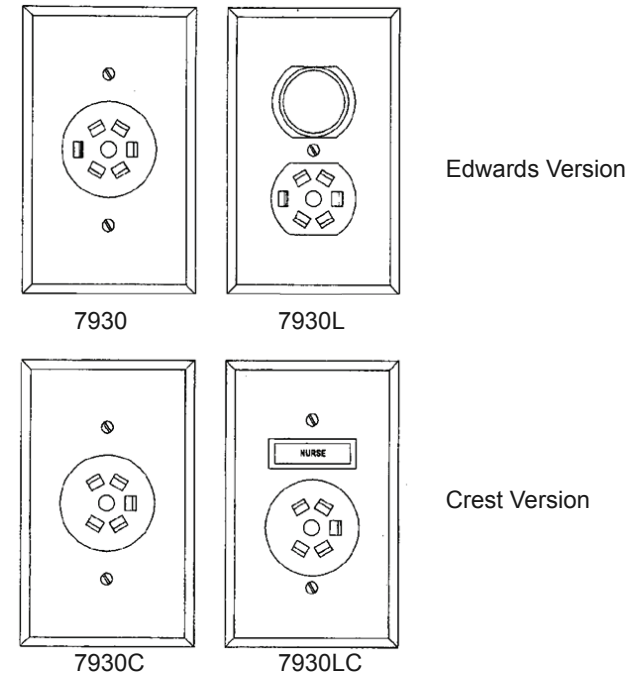

Typical Installation

1. Review designations on the Main Terminal: 1, 2, 3, B and C.
2. Note these designations on the PCB of Crest replacement station. (See Figure 1)
3. Screw wires from the Nurse terminals to Crest replacement station.
4. Cap any unused wires.
5. Always test the nurse call station for correct operation (including cord out) before placing into service.

Specifications

ELECTRICAL	
Voltage Rating:	24VAC input
MECHANICAL	
Connector:	Screw
Wire Termination:	Screw
Mounting:	Fits standard single gang back box
SIZE MEASUREMENTS	
Plate:	4.5" x 2.75" (1 Gang)

Electrical Schematic

