



8721C



8723C



8710C

Bedside Station – Crest Replacement for Jeron®

8721C, 8723C & 8710C

Typical Installation

1. Replace existing station with replacement, wire for wire.
2. Fasten eyelet of green lead to the backside of the faceplate through the mounting hole when screwing station to wall box.
3. Always test all nurse call stations for correct operation (including cord out for 8721C & 8723C) before placing into service.

Specifications

ELECTRICAL

Voltage Rating& Current: 24VDC @ 20mA

MECHANICAL

Terminals: Wire Leads

Mounting: Fits standard single gang back box

SIZE MEASUREMENTS

Plate: 4.5" x 2.75" (1 Gang)

Function Table

Jeron Original Wire Color	Function
White/Green	(A) Output
White/Orange	(B) Output
Brown	(-) 24VDC
Orange	(+) 24VDC
Green	Ground