



HVB-100BC



HVB-201BC

Single & Dual Patient Stations, Crest Replacements for Ektacom®

P/N HVB-100BC (Single)
 HVB-201BC (Dual)

Typical Installation

1. Review wire designations on the old station.
2. Replace the old station with Crest replacement station wire for wire.
 Note: Review Wire Color Comparison table for reference.
3. All unused wires must be capped.
4. Always test nurse call station for correct operation before placing into service.

Specifications

ELECTRICAL	
Voltage Rating:	24 VDC input
MECHANICAL	
Terminals:	Wire Leads
Mounting:	Fits on standard single/dual gang back box
SIZE MEASUREMENTS	
Plate:	4.5" x 2.75" (Single Gang) 4.5" x 4.562" (Dual Gang)

Wire Color Comparison Table

Executone	Crest
Violet	Violet
Green	Green
Red	Red (+24 VDC)
White	White (-24 VDC)

Schematic (Entire Circuit Not Shown)

