




PST



Wall Mount Privacy Screen

TOOLS REQUIRED:

- Tape measure
- Electric drill
- 1/8" drill bit
- Phillips screwdriver

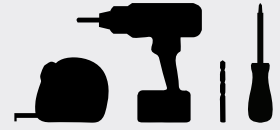

1 person assembly

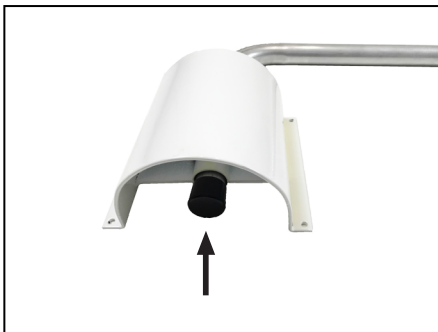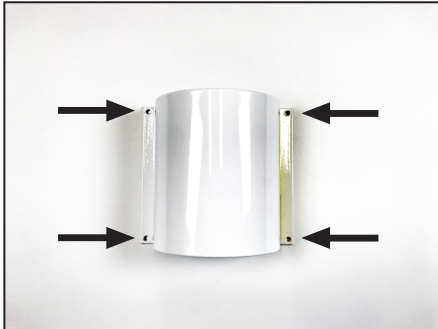

STEP  
1

The recommended mounting height for the bracket is 80" from the floor to the top of the bracket. Locate wall stud and align one side of the bracket so the screws hit the wall stud. Mark all four holes on the wall (center mark each hole with a pencil or pen). Using 1/8" drill bit, make pilot holes for each screw. Fasten one side directly to the stud. Use wall anchors for the other side of the bracket. If no wall stud is available, use all four anchors and screws to mount the bracket.

STEP  
2

After mounting bracket to wall, insert the short end of the telescoping rod into the hole in the bracket. The expandable arm should move easily from side to side.

WARNING



- Do not hang anything from the rod other than the curtain.
- Do not use the rod as an aid to assist a patient.

HARDWARE INCLUDED (Not to scale)

- |     |                         |    |              |    |              |
|-----|-------------------------|----|--------------|----|--------------|
| x13 | Curtain rings           | x4 | Screws       | x4 | Flat washers |
| x1  | Cover slide - stop ring | x4 | Wall anchors | x1 | Plastic cap  |





PST

Wall Mount Privacy Screen

pg 2



STEP  
3

Roll the rubber slide-stop ring onto the telescoping rod so it is on the horizontal tube, about 1" away from the bend, or where you want the curtain to stop sliding on the rod.



STEP  
4

Place all 13 curtain rings onto the rod. Attach each ring to a grommet on the curtain and fasten each one closed.



STEP  
5

Your telescoping privacy screen is ready for use. To clean the vinyl panel, use mild liquid soap and water, allow it to drip dry or towel dry.

