
# **Remote TENS Pain Management Solution**

22-041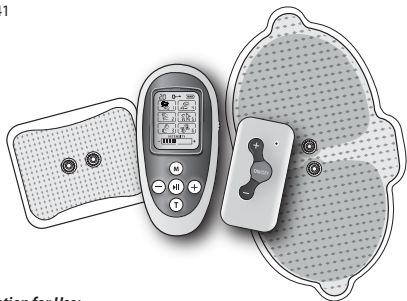

***Intention for Use:***

***The Remote TENS Pain Management Solution is intended for temporary relief of pain associated with sore and aching muscles due to strain from exercise or normal household and work activities.***

**PLEASE READ THESE INSTRUCTIONS COMPLETELY BEFORE USING YOUR TENS DEVICE**

**INSTRUCTION MANUAL • ENGLISH & ESPAÑOL**


**STOP! PLEASE ENSURE YOU HAVE ALL OF THE FOLLOWING COMPONENTS BEFORE USING YOUR TENS DEVICE**

Remote Control

Electrical  
Stimulator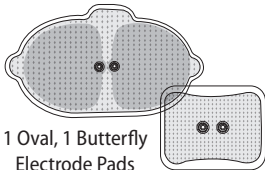

1 Oval, 1 Butterfly  
Electrode Pads

USB Cable

Storage Bag

Instruction Manual  
Replacement  
Electrode Order Form

**IF YOU ARE MISSING ANY PARTS, INCLUDING INSERTS OR INSTRUCTION MANUALS, DO NOT RETURN TO PLACE OF PURCHASE. CONTACT CUSTOMER CARE AT 866-326-1313.**

---

# INDEX

---

Care and Safety Information.....	4-11
Product Features.....	12-13
Charging the Device.....	14-15
Preparing Your Device.....	16
Placement & Treatment.....	17
Pulse Program Descriptions.....	18-19
Selecting Your Gel Pad.....	19
Instructions for Use.....	20-23
Caring for Electrode Pads.....	24
Cleaning and Storage.....	25
Troubleshooting.....	26
Product Specifications.....	27
Warranty Information.....	29-29
Español.....	31-58

---

**Toll-Free Customer Care Help Line: 1-866-326-1313**  
**Monday – Friday 8:30 a.m. – 4:30 p.m. CST**

Distributed by  
Veridian Healthcare  
1175 Lakeside  
Gurnee, IL 60031  
[www.veridianhealthcare.com](http://www.veridianhealthcare.com)

Made in China

#93-1279 11/16  
©2016 Veridian Healthcare, LLC

---

# IMPORTANT SAFEGUARDS WARNINGS & CAUTIONS

---

**NOTE:** Read all instructions and inserts included with this device carefully before use. The following basic precautions are needed when using an electrical product.

**CAUTION:** Failure to read and observe all precautions could result in injury or equipment damage.

Improper care or use of your device may result in injury, damage to the unit or ineffective treatment. Following these instructions will ensure the device's efficacy and long life.

## **CONTRAINDICATIONS :**

Do not use this device if you have a cardiac pacemaker, implanted defibrillator or other implanted metal or electronic device.

Do not use this device to treat undiagnosed chronic pain.

Do not use this device if you have suspected or diagnosed heart problem.

Do not use this device if you have suspected or diagnosed epilepsy.

Do not use this device if you have a tendency to bleed internally following an injury.

Do not use this device if you recently had surgery, or have ever had surgery on your back.

---

# IMPORTANT SAFEGUARDS WARNINGS & CAUTIONS

---

Do not use this device if areas of skin lack normal sensations, such as skin that tingles or is numb.

Do not use this device during pregnancy.

## GENERAL WARNINGS PRIOR TO USE

- If you are under the care of a Physician, consult with your Physician before using this device
- Long-term effects of this device are not known
- Do not place the device on or close to your heart
- Take care when applying device close to the neck; do not place this device on the front or sides of the neck. Severe spasm of the muscles may occur and the contractions may be strong enough to close the airway or cause difficulty in breathing. Stimulation over the neck could also affect hearing or blood pressure
- Do not apply stimulation across the chest because the introduction of electrical current into the chest may cause rhythm disturbances to the heart
- Do not place the device on or around your head. The effects of stimulation of the brain are unknown

---

# IMPORTANT SAFEGUARDS WARNINGS & CAUTIONS

---

- Do not use this device while sleeping
- Do not use if you feel numbness
- Do not use this device in or close to water
- Use the only on normal, healthy, clean and dry skin. Do not use on or close to open wounds or rashes, or over swollen, red, infected or inflamed skin
- Do not apply stimulation over, or in proximity to, cancerous lesions
- Do not use the device on children or incapacitated persons
- Consult with your physician before using this device, because the device may cause lethal rhythm disturbances to the heart in susceptible individuals

## **GENERAL CAUTIONS PRIOR TO USE**

- Read this manual before using this device for the first time
- Keep this manual available whenever you use this device
- This device is intended for individual person use only
- This device is not effective for pain associated with Central Pain Syndromes, such as headaches

---

## **IMPORTANT SAFEGUARDS WARNINGS & CAUTIONS**

---

- This device is for pain caused by muscle soreness, and should be placed only around muscles where pain originates
- The pain may indicate that you have some other health problem. You should know the reason and source of your pain before using these devices. Do not solely rely on the treatment of this device for pain
- The safety of using this device during pregnancy or birth has not been established
- The effectiveness of this device depends greatly on a person's individual physical condition. It may not always be effective for every user
- If you have had medical or physical treatment for your muscle pain, consult with your treatment provider before using this device. You should contact your physician prior to using this device following recent surgical procedures. Stimulation may disrupt the healing process

---

# IMPORTANT SAFEGUARDS WARNINGS & CAUTIONS

---

## OPERATING WARNINGS & CAUTIONS

- CONSULT WITH YOUR PHYSICIAN IF YOU HAVE ANY OF THE FOLLOWING:
  - If you have suspected or diagnosed heart problem
  - If you have suspected or diagnosed epilepsy
  - If you have a tendency to bleed internally following an injury
  - If you recently had surgery, or have ever had surgery on your back
  - If areas of skin lack normal sensations, such as skin that tingles or is numb
  - During menstruation or during pregnancy
  - The unit is intended for the temporary relief of pain caused by muscle soreness. The source or cause of the pain should be addressed by a healthcare professional. Pain could indicate an injury or condition that requires treatment
  - Some people may feel skin irritation or experience a very sensitive feeling in the skin due to electrical stimulation. If this occurs, stop using these devices and consult your physician

---

# IMPORTANT SAFEGUARDS WARNINGS & CAUTIONS

---

- If skin under the pad feels irritated after using the stimulator for a long period of time, use the stimulator for a shorter period of time
- Minor redness at stimulation placement is a normal skin reaction. It is not considered a skin irritation, and it will normally disappear within 30 minutes after the electrodes are removed. If the redness does not disappear after 30 minutes from the removal of electrodes, do not use the stimulator again until after the excessive redness has disappeared
- Turn off your device if the stimulation feels unpleasant or does not provide pain relief
- Keep this device out of the reach of children.
- Use this device only with the pads and accessories recommended by the manufacturer
- Do not use this device when driving, operating machinery or when swimming
- Before removing the Electrode Pad, be sure to turn it OFF, avoiding unpleasant stimulation
- TENS is not a substitute for pain medications and other pain management therapies

---

## **IMPORTANT SAFEGUARDS WARNINGS & CAUTIONS**

---

- TENS devices have no curative value
- TENS is a symptomatic treatment and, as such, suppresses the sensation of pain that would otherwise serve as a protective mechanism
- Do not use the unit while sleeping
- After each use, clean any adhesive residue left on the skin with soap and water
- The effectiveness of stimulus therapy varies from person to person. It may not be an effective treatment for all users
- The long-term effects of stimulus therapy are unknown

## **STORAGE WARNINGS & CAUTIONS**

- Storage outside of stated storage temperature may result in measurement error or device malfunction; storage environment temperature is: 14°F – 140°F (-10°C – 60°C); humidity: 30-95% RH
- Keep the unit out of reach of small children

---

## **IMPORTANT SAFEGUARDS WARNINGS & CAUTIONS**

---

- Do not store the device in direct sunlight, dusty or humid environments, or extreme temperatures. Exact figures for appropriate use and storage can be found in the Product Specifications section of this manual

### **CLEANING WARNINGS & CAUTIONS**


- Never immerse the unit in water to clean as it may damage the unit
- Follow the 'Cleaning and Maintenance' portion of this manual for instruction on how to clean and care for your device

---

# PRODUCT FEATURES


---

## *Remote Control*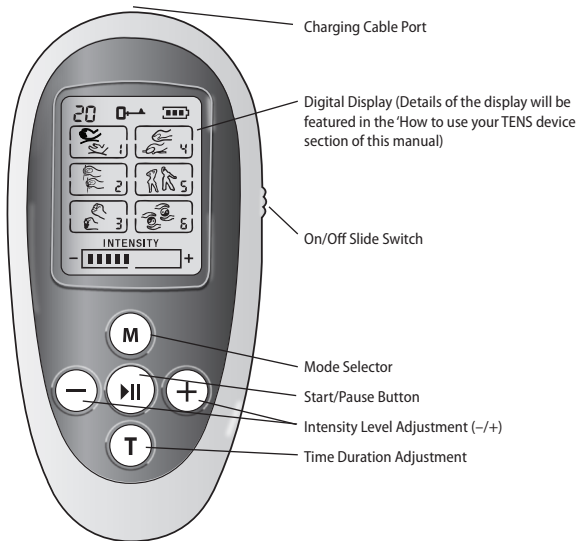


# PRODUCT FEATURES

## *Electrical Stimulator*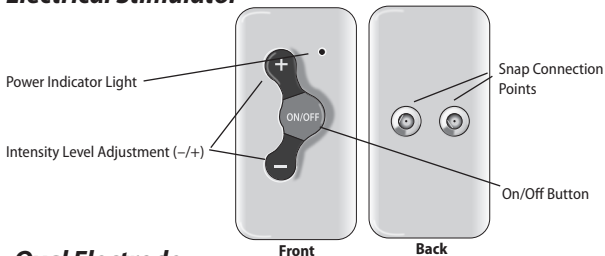

## *Oval Electrode Pad*

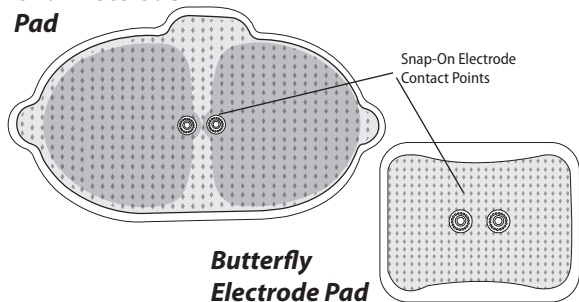---

## CHARGING THE DEVICE


---

**This device includes a rechargeable battery in the remote control and electrical stimulator unit as well as USB cable.**

Before first use, the devices should be charged for approximately 2 hours. Subsequent charging will also take approximately 2 hours.

The Remote Control and Electrical Stimulator Unit are charged following the same steps:

1. Connect the wide end of the USB cable to a wall adapter (not included).
2. Connect the smaller end to the selected device.
3. Each piece has a unique feature to indicate that charging is complete, as illustrated on the facing page.

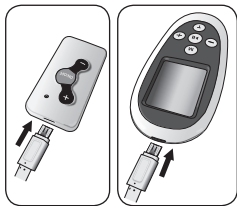# CHARGING THE DEVICE

During charging the power indicator light will flash green.

When charging is complete the indicator light will be solid green.

Unplug the wall charger when the charging cycle is complete.

## NOTE:

A full charge will power 20 twenty-minute treatment cycles at a medium intensity setting for the electrical stimulator and 30 twenty-minute treatments for the remote control before needing to be recharged.

During charging battery icon on the upper right of the display will flash. 

When charging is complete the battery icon will be solid.

Unplug the wall charger when the charging cycle is complete.

---



## PREPARING YOUR DEVICE

---

It is necessary to prepare and assemble your device prior to treatment.

**NOTE: Save the plastic film discs for storage after treatment to protect the pads from dirt and debris.**

1. While the stimulator device is OFF (the green power light is NOT illuminated) carefully attach the electrode pads to the control unit via the snap connection points.
2. Pull gently to ensure the electrode pad is firmly attached.
3. Carefully remove the plastic film from the electrode pad.
4. You are ready to apply the pad to the affected treatment area.

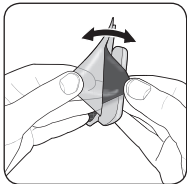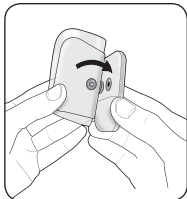**Do not turn the power ON until the device is in place on the user's skin as indicated in the 'Instructions for Use' section of this manual.**

---

# PLACEMENT & TREATMENT

---


## **Read all warnings and cautions completely before use.**

The electrode pads are applied directly to the skin. Clothing may be worn over the device; clothing should not constrict the device and should allow access to the control unit buttons.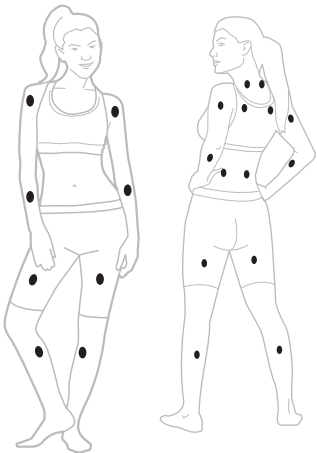

Targeted treatment areas may include:

- shoulder
- elbow
- thigh
- knee
- calf
- ankle
- back of neck
- upper back
- lower back

**Never place device on over the heart, on the front or sides of neck or the face.**

# PULSE PROGRAM DESCRIPTIONS

	<b>Pulse Rate</b>	<b>Description</b>	<b>Benefit</b>
P1 	65	Pulse on for 3.4 second and off for 1.6 second	Mild gate pain relief
P2 	13-56	Pulse on for 20 second and off for 1 second	Muscle performance improvement with variable pulse rate
P3 	1.2	Pulse on every 850 milliseconds	Slow endorphin pain relief
P4 	100	Pulse on for 3.6 second and off for 1 second	Quick gate pain relief and strong pulse amplitude
P5 	100	Pulse on for 10 second and off for 2.5 second	Quick gate pain relief
P6 	Combina- tion P1-P5	Combination of the above	All kinds of pain relief

---

## PULSE PROGRAM DESCRIPTIONS

---

The programs are not specific to treatment at a certain body location or to treat a specific ailment or pain.

The sensations may include—massage, acupuncture, tapping, scraping or a combination of the sensations at variable durations and cycles.

Select the program that provides the best relief for your particular ache and pain.


---

## SELECTING YOUR GEL PAD

---

This device includes two sizes of gel electrode pads. There is no functional difference in the pad. Select the size for use most appropriate to the size of the treatment area.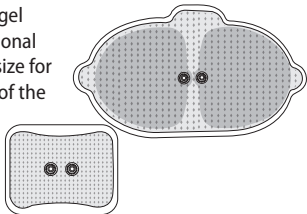

---

## INSTRUCTIONS FOR USE

---

**Read all warnings and cautions completely before use. Recommended treatment time is 20-minutes; consult your healthcare professional for a treatment cycle that best suits your injury. Discontinue use immediately if treatment is uncomfortable or painful.**

1. Treatment area should be clean before applying the control unit. Note that excessive skin lotion or soap residue may affect the adhesiveness of the pad. Only use on HEALTHY skin areas with no cuts, sores or wounds.
2. Apply the pad to the desired treatment area, taking care to ensure the controller is comfortably within reach of your hand for safety and control during treatment.
3. Ensure the electrode pad is smoothly applied and the gel adhesive is holding the pad firmly in place on the skin.


---

# INSTRUCTIONS FOR USE

---

## Instructions for Use **WITHOUT** the Remote Control


4. This device may be used at the default program 1 without the remote control; the remote must be used to access other programs.
5. After the device is applied to the desired treatment area, press the **ON/OFF** button; the green light will illuminate.
6. The TENS unit will begin to operate at the lowest intensity level setting for program 1.
7. Press and release the '+' button to find your desired therapy sensation level. This device offers 20 intensity levels. Press and release the '-' button to reduce the intensity level.
8. The TENS unit will automatically shut off after 20 minutes of operation. Press the **ON/OFF** button to turn the device off at any time during treatment.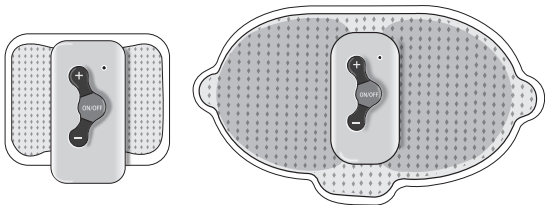

# INSTRUCTIONS FOR USE

## Instructions for Use WITH the Remote Control

- After the device is applied to the desired treatment area, press the **ON/OFF** button; the green light will illuminate.
- The power switch is located on the side of the remote control; slide the switch to the **ON** position and the digital display will illuminate.

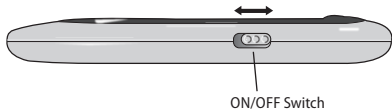- To change the mode, while the unit is on/operating, press and release the 'M'; the mode will switch to the next in the sequence of 6 pulse types. The current mode in use will flash on the display. See page 18 for a detailed description of each pulse type. When changing modes, the intensity will revert to level 1 (lowest sensation).

- The sensation should be strong but comfortable; press the + or - button at any time during the treatment to change the intensity as desired.

---

# INSTRUCTIONS FOR USE

---

**Until you are familiar with use of the unit, begin with the lower intensity levels and gradually progress to more advanced levels as you feel comfortable.**

---

 The key icon will remain solid while the device is active, and flash while it is paused.

 The intensity level is immediately visible at a glance during use.

---

- The timer default treatment time is 20-minutes. To adjust the time in 10 minute increments, press and release the 'T' button to scroll to the desired time up to 60 minutes.
- To discontinue use prior to the end of treatment time, press **PLAY/PAUSE (>II)** button or slide the power to the OFF position.
- Be sure the device is OFF (no sensation is felt) before removing the device from your skin. Take care during removal to best protect the adhesive and preserve the life of the electrode pad.
- Place the plastic cover over the gel side of the pad to prevent them from drying out.
- To conserve the battery and prevent shocks, the device will automatically power off when not in use.

## **NOTE:**

You may wish to clean the treatment site after use to remove any adhesive residue from the electrode pad.

---

---

# CARING FOR THE ELECTRODE PADS

---

## **Gel electrode pads**

- The electrode pads can be reused as long as they retain their stickiness. Cover the pads with the included film pieces when not in use to prevent them from drying out
- Properly cared for, the electrode pads may last up to 50 uses; the life of the electrode pad is largely dependent upon proper care and the condition of the skin as applied for use
- To help prolong the life of the electrode pad, between uses, moisten the gel side of the pad with a drop of water and allow to dry; replace the film for storage
- If you notice that the pads have become dry, no longer adhere well to the skin, or begin to irritate the skin, they are in need of replacement.

Replacement pads are available through Veridian Healthcare at 866-326-1313, by using the enclosed order form or through many major retailers and ecommerce sites.

---

# CLEANING AND STORAGE

---

## Cleaning

- Unsnap the electrode pad before cleaning the device
- The Electrical Stimulator and Remote Control can be cleaned with a soft, dry cloth
- Never use cleaning agents or excessive water to clean any part of the unit. Do not submerge any part of the unit or let it get wet
- Never disassemble the unit or attempt to repair the control unit

## Storage

- Always use the included plastic film to protect the gel electrode from dust and debris
- If the device will not be used for several days or longer, place the film cover over the electrode pads to prevent drying out or getting dirt and debris on them
- Store all components in the included bag while not in use
- Storing the pads in a plastic zip bag will help prevent the gel pads from drying out prematurely
- Protect the unit from mechanical shock or heavy impact
- Avoid exposure to extreme temperatures

---

# TROUBLESHOOTING

---

<b>Problem</b>	<b>Solution</b>
Display does not illuminate/ is weak / Control unit will not turn on	Recharge the batteries on both the Electrical Stimulator and Remote Control
Intensity output is low / no stimulation is occurring	Electrode pad may be dirty; clean and attempt again. Confirm that the electrode snaps are firmly connected to the control unit. Battery power is low, recharge the battery. Device may not be applied properly; ensure the gel pad is making complete contact with the skin.
Electrode pads are not adher- ing to the skin	Review the 'Caring for Electrode Pads' section of this manual; pads may need to be replaced

---

# PRODUCT SPECIFICATIONS

---

<b>NAME</b>	Remote TENS Pain Management Solution
<b>MODEL NUMBER</b>	22-041
<b>CHANNEL</b>	Single Channel
<b>OUTPUT VOLTAGE</b>	0~42V +/- 20% (at 500Ohm load)
<b>PULSE INTENSITY</b>	0~84mA +/- 20%; 0~20 stages adjustable
<b>PULSE WIDTH</b>	0~100 uS
<b>PULSE RATE</b>	0 ~ 200Hz
<b>AUTOMATIC SHUT-OFF</b>	After completion of set therapy treatment time; 10–60 minutes maximum
<b>POWER SOURCE</b>	Rechargeable Lithium Polymer Battery
<b>OPERATION ENVIRONMENT</b>	Temperature 50 F - 104 F (-10 C - 40C); Humidity 30% - 85%
<b>STORAGE ENVIRONMENT</b>	Temperature 14 F - 140 F (-10 C - 60 C); Humidity 30% - 95%
<b>DEVICE DIMENSIONS</b>	3-1/2" x 1-5/8" x 3/8"
<b>WEIGHT</b>	Controller weight 1.3 oz
<b>INCLUDED</b>	Electrical stimulator, remote control, large oval gel electrode pad, butterfly gel electrode pad, USB cable, instruction manual, storage bag, replacement electrodes order form
<b>AVAILABLE SEPARATELY</b>	Replacement electrodes 2-pack, 22-047

*Specifications are subject to change without notice*

---

# 1-YEAR LIMITED WARRANTY

---

Congratulations on your purchase of the TENS Pain Management Solution. Your TENS Pain Management Solution is covered by the following limited warranty commencing upon the date of purchase, and subject to the following terms and conditions:

The Warrantor warrants that its TENS Pain Management Solution will be free from defects in materials and workmanship under normal consumer usage for a period of one year for the original purchaser of the product.

Periodic maintenance, repair and replacement of parts due to normal wear and tear are excluded from coverage. Defects or damage that result from: (a) improper operation, storage, misuse or abuse, accident or neglect, such as physical damage (cracks, scratches, etc.) to the surface of the product resulting from misuse; (b) contact with liquid, water, rain, extreme humidity or heavy perspiration, sand, dirt or the like, extreme heat, or food; (c) use of the TENS Pain Management System for commercial purposes or subjecting the TENS Pain Management System to abnormal usage or conditions; or (d) other acts which are not the fault of the Warrantor, are excluded from coverage. This warranty does not cover batteries or other power sources that may be provided with, or used with the TENS Wired Pain Management Solution.

If the TENS Pain Management Solution fails to conform to this limited warranty, return the TENS Pain Management Solution postage prepaid to: **Attn: Repair Department, 1175 Lakeside Drive, Gurnee, IL, 60031**. When returning a product, please also include: (i) a copy of your receipt, bill of sale or other comparable proof of purchase; (ii) a written description of the problem; and (iii) your name, address and telephone number. Carefully package the product

---

# 1-YEAR LIMITED WARRANTY

---

to avoid any damage that may occur while in transit; shipping insurance with returned receipt is recommended. At our option, the Warrantor will repair or replace the unit found to be defective in materials or workmanship under normal consumer usage. The purchaser will be notified of any additional repairs required prior to completing the repair, and will be responsible for parts charges, if any, and repair charges not covered by this limited warranty.

EXCEPT AS PROVIDED FOR IN THIS LIMITED WARRANTY, ALL EXPRESS AND IMPLIED WARRANTIES AND CONDITIONS ARE DISCLAIMED, INCLUDING WITHOUT LIMITATION THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THE REPAIR OR REPLACEMENT AS PROVIDED UNDER THIS LIMITED WARRANTY IS THE EXCLUSIVE REMEDY OF THE CONSUMER, AND IS PROVIDED IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. IN NO EVENT SHALL THE WARRANTOR BE LIABLE, WHETHER IN CONTRACT OR TORT (INCLUDING NEGLIGENCE) FOR DAMAGES IN EXCESS OF THE PURCHASE PRICE OF THE PRODUCT, OR FOR ANY INDIRECT, INCIDENTAL, SPECIAL OR CONSEQUENTIAL DAMAGES OF ANY KIND, OR FOR DAMAGES TO, OR LOSS OF, OTHER PROPERTY OR EQUIPMENT OR PERSONAL INJURIES TO THE FULL EXTENT THESE DAMAGES MAY BE DISCLAIMED BY LAW.

Some states and jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitation on the length of an implied warranty, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or from one jurisdiction to another.

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