by Veridian Healthcare

Soothing Gel Eye Mask

Model: 24-809

Non-Toxic • Use only as directed

DIRECTION FOR USE:

Always inspect pack for cracks or tears before use. Do not exceed recommended therapy time of 20 minutes. CONSULT THE BELOW INSTRUCTIONS FOR DETAILED WARNINGS AND CAUTIONS. Monitor skin during use.

COLD THERAPY

Place gel pack in refrigerator or freezer for a minimum of 1 hour before use. Gel pack may be stored in refrigerator or freezer for immediate use.

CAUTION:

It is recommended that you wrap the compress in a towel or cloth before application. For external use only. Do not use bag if it is punctured, torn or leakage occurs.

WARNING:

Use carefully. May cause serious burns. Do not use over sensitive skin areas or in the presence of poor circulation. Extreme cold may cause frostbite. Stop treatment immediately and seek medical attention for a reaction to skin, pain or severe discomfort. The unattended use of Gel Eye Mask by children, elderly or incapacitated persons may be dangerous. Cold therapy is not recommended for people with circulatory problems, Raynauds Syndrome or diabetes without consent by your physician.

If gel comes in contact with eyes or open wound, flush with water. If gel comes in contact with skin, remove any contaminated clothing and wash the area with mild soap and water. Seek medical attention for any persistent symptoms including irritation or redness.

This product is non-toxic. Discard if bag becomes damaged or leaks. Do not mix with combustible materials.

**Toll-Free
Customer Care Help Line:
1-866-326-1313
Monday – Friday
8:30 a.m. – 4:30 p.m. CST
www.veridianhealthcare.com**

Manufactured for:
Veridian Healthcare
1175 Lakeside Drive
Gurnee, IL 60031
www.veridianhealthcare.com

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©2019 Veridian Healthcare, LLC