ProKit™ Combo **with Attaché Case**

Includes Sprague Rappaport-Type Stethoscope

Model 02-126 Series

**Please read this instruction manual completely
before operating this unit.**

INSTRUCTION MANUAL • ENGLISH & ESPAÑOL

STOP!
**PLEASE ENSURE YOU HAVE ALL OF THE
FOLLOWING COMPONENTS BEFORE USING
YOUR PROKIT™ COMBO.**

Sprague
Rappaport-Type
Stethoscope

Gauge


Squeeze Bulb

Adult
Arm Cuff

Storage Case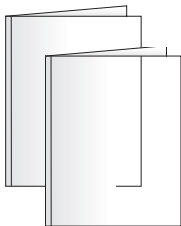

Instruction
Manual

**IF YOU ARE MISSING ANY PARTS, INCLUDING INSERTS OR
INSTRUCTION MANUALS, DO NOT RETURN TO PLACE OF
PURCHASE. CONTACT CUSTOMER CARE AT 866-326-1313.**

INDEX

Care & Safety Information	4-5
Introduction	6
Getting Started	7
Features & Assembly.....	8
Applying Your Blood Pressure Cuff.....	9
Proper Stethoscope Use & Fit	10
Stethoscope Replacing Parts	11-12
Taking Your Blood Pressure Reading	13-14
Care & Maintenance.....	15
Device & Label Symbols.....	16
Sphygmomanometer Warranty	17
Stethoscope Warranty.....	18
Instrucciones en Español.....	19-36

Toll-Free Customer Care Help Line: 1-866-326-1313
Monday – Friday 8:30 a.m. – 4:30 p.m. CST

Manufactured for
Veridian Healthcare
1175 Lakeside Drive
Gurnee, IL 60031
www.veridianhealthcare.com

Made in China

#93-1475 05/20
©2020 Veridian Healthcare, LLC

CARE & SAFETY INFORMATION

CAUTION: Failure to read and observe all precautions could result in personal injury or equipment damage.

Improper care or use of your sphygmomanometer may result in injury, damage to the unit or ineffective treatment. Following these instructions will ensure the device efficacy and long life.

GENERAL CAUTIONS AND WARNINGS

- Measurement results should be discussed with your physician or healthcare professional; never self-diagnose or attempt treatment as this may be dangerous.
- Intended for adult use only; this device is not approved for neonate, infant or child use. Not for use by pregnant women.
- Keep out of reach of children.
- The patient is the intended operator of this device.
- Do not use the unit if it has any damaged parts if it has been submersed in water or dropped.
- If any abnormality occurs, discontinue use until the unit has been examined and repaired.
- Only use included attachments and parts; do not use attachments from other brands or models as these may not be properly calibrated for use with this device and may result in measurement error.
- Do not disassemble the unit or attempt to repair it; substitution of a component different from that supplied may result in measurement error and will void manufacturer warranty.
- Always follow local regulations for proper disposal.
- Do not share this device with infected persons where there may be a risk of cross-contamination.
- Consult with your physician before if the cuff may be applied over a wound or inflammation diseases.
- Consult with your physician before use if any of the following conditions are present: arrhythmia, common arrhythmias such as atrial or ventricular premature beats or atrial fibrillation, arterial sclerosis, poor perfusion, diabetes, pre-eclampsia, renal diseases.

CARE & SAFETY INFORMATION

inflammatory diseases, intravascular therapy, arteriovenous shunts or if a mastectomy has been performed.

- Motion, trembling and shivering may affect measurement reading.
- If you are allergic to plastic or rubber, consult with your medical professional before using this device.

OPERATING CAUTIONS AND WARNINGS

- Prolonged over-inflation may cause congestion, swelling or bruising in some people.
- Too-frequent measurements may cause injury due to blood flow interference.
- Check the circulation of the user before, during and after use.
- If you experience any discomfort or an abnormality occurs, stop using the device immediately.
- Never use this unit while operating a vehicle or in the bathtub or shower.

STORAGE CAUTIONS AND WARNINGS

- Storage outside of stated storage temperature may result in measurement error or device malfunction; storage environment temperature is: -50°F – 104°F (-10°C – 40°C); Humidity: 15-85%RH.
- Keep the unit out of reach of children.

CLEANING CAUTIONS AND WARNINGS

- Never immerse the unit in water to clean as it may damage the unit.
- Follow the 'Care & Maintenance' portion of this manual for instruction on how to clean and care for your device.

INTRODUCTION

It is recommended that you first seek the advice and recommendation of your physician or healthcare professional when using home diagnostic devices, including blood pressure monitors.

Thank you for purchasing a ProKit™ Combo with Sprague Rappaport-Type Stethoscope. With proper care and use, your kit will provide you with many years of reliable measurements.


**PLEASE READ THIS INSTRUCTION MANUAL
COMPLETELY BEFORE OPERATING THIS UNIT.**

GETTING STARTED

Before Taking a Measurement

1. Your readings should only be interpreted by your physician or healthcare professional with access to your individual medical history. Regular home blood pressure monitoring will allow you to track and record your readings for discussion with your physician.
2. Conduct your measurement in a quiet place while seated in a relaxed position. Rest for 15 minutes before taking your reading. If an error occurs or you desire to take a second reading, allow 15 minutes between readings for your blood vessels to return to normal.
3. Avoid smoking, eating, taking medication, alcohol, physical activity or any other stressful activity for 30 minutes prior to taking a reading.
4. Always remove any jewelry or constrictive clothing that may interfere with the cuff placement.
5. Remain still during measuring; do not talk during the reading.
6. It is recommended that you take your readings as the same time each day to better monitor any indications in your results.
7. Record your measurements to share with your physician.

FEATURES & ASSEMBLY

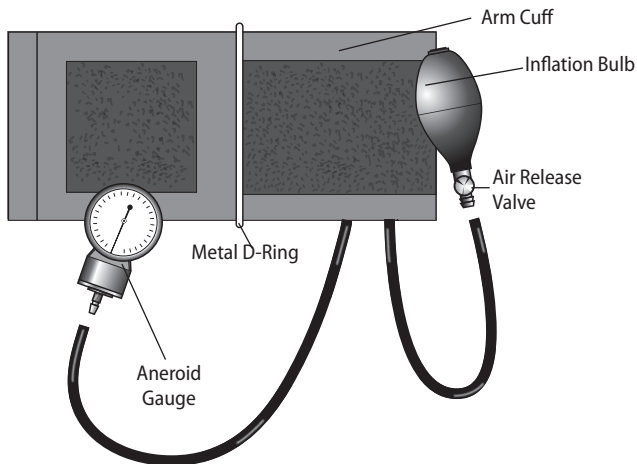Simple Assembly -

Cuff, Bulb and Gauge: Attach the gauge and bulb to the tubing as shown above.

APPLYING YOUR BLOOD PRESSURE CUFF

Before applying your blood pressure cuff, be sure you have selected the appropriate size cuff:

Adult Cuff fits arm circumference 11" - 16.375" (27.9-41.7 cm)

If for any reason you are unable to or should not use your left arm, please modify the instructions for cuff application to your right arm. Your physician can help you identify which arm is best for you.

1. Remove any constrictive clothing or jewelry that may interfere with cuff placement.
2. Be seated at a table or desk with your feet flat on the floor.
3. Extend your left arm. Find your brachial artery, which is at the inner aspect (bend) of your elbow, Fig 1. You may be able to feel your pulse. The artery patch should point to the brachial artery when the cuff is applied.
4. Open the cuff creating a circle that you can slide your arm into; do not extend the cuff beyond the metal bar.
5. Insert your left arm into the cuff. Position the bottom edge of the cuff approximately one-half inch ABOVE your elbow; when taking your reading, the stethoscope chestpiece must directly touch the inside of your arm, over your brachial artery.
6. With your arm extended, palm facing upward and tubes positioned down the center of your arm, Fig 2-3, use your right hand to pull the end of the cuff and secure the hook and loop material. Make sure that the artery mark is still pointed to your brachial artery.
7. The cuff should fit comfortably, yet snugly around your arm. You should be able to insert one finger easily between your arm and the cuff.

Fig. 1

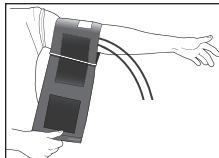
Fig. 2

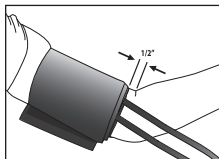
Fig. 3

If you are not comfortable with applying your cuff, please seek the assistance of another member of your household or work with your physician to practice the cuff application. Incorrectly applied cuffs may result in inaccurate readings.

PROPER STETHOSCOPE USE & FIT

The earpieces on your stethoscope have been carefully calibrated to provide clear and audible acoustics. To best hear the sounds to be analyzed, please keep the following tips in mind:

The earpieces should point forward when in use, Fig 1. The earpieces should fit in ears securely but comfortably.

The binaurals may be adjusted to ensure maximum comfort and efficacy. To increase the tension, simply squeeze the binaurals just above the joint where they meet, crossing the earpieces, Fig 2. Do not apply too much pressure; apply a minimum of force, slightly increasing the pressure until the earpieces fit comfortably.

Should the earpieces be too tight, pull the binaurals apart, grasping them just above the joint where they meet, Fig. 3. Be gentle so as not to damage the stethoscope.

Always use caution when adjusting the binaurals to avoid damage. When bringing the earpieces closer together, the maximum tension used should bring the black tubing on either side within contact. When pulling the binaurals apart, do not exceed 90 degrees between the binaurals.

Fig. 1 -
Correct Position

Fig. 1 -
Incorrect Position

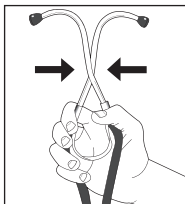
Fig. 2

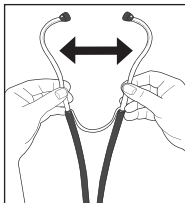
Fig. 3

STETHOSCOPE REPLACING PARTS

Note: Use only Veridian replacement parts in this unit, available from your retailer or directly from Veridian Healthcare at 866-326-1313.

Diaphragm and Bell Use:


Large Diaphragm - Detects diastolic, first, second and third heart sounds, low frequency heart sounds and high-pitched murmurs in adults.

Small Diaphragm - For high-pitched infant or pediatric heart sounds.


Large Bell - Pinpoints low and medium pitched sounds and murmurs in adults.

Medium Bell - Detects low and medium pitched sounds or murmurs in narrow or small places.

Small Bell - For low and medium pitched sounds and murmurs from an infant or pediatric patient.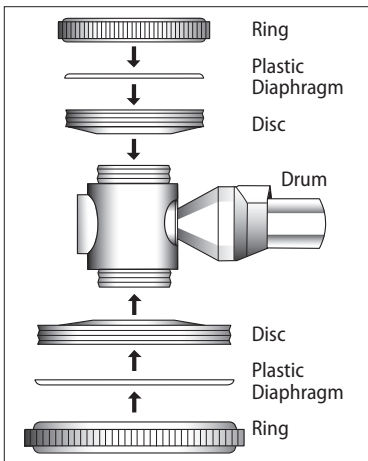

The head clicks into position with each 180° turn.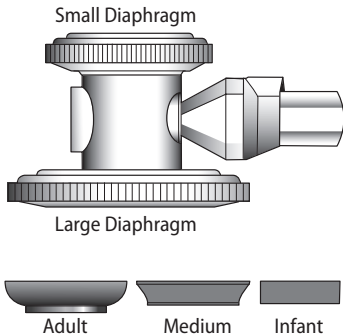

STETHOSCOPE REPLACING PARTS

Changing the Diaphragm:

Sprague Rappaport-Type Stethoscopes include a chestpiece key for easy diaphragm replacement, fig 1.

Remove either the small or large diaphragm from the chestpiece.

Place the chestpiece diaphragm side down on the palm of your hand. Using the key, twist and remove the inner metal disc for access to the plastic diaphragm, fig 2.

After replacing the diaphragm, use the metal key to tighten the metal disc into place and reattach to the chestpiece, fig 3.

Changing the Components:

Anytime a component is changed, tap one side of the chestpiece to confirm that the stethoscope is functioning properly. If no sound is heard through the binaural tips, rotate the head 180° or until a clicking sound is heard. Repeat tap.

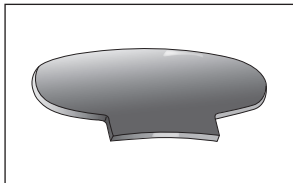
Fig. 1

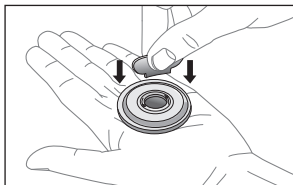
Fig. 2

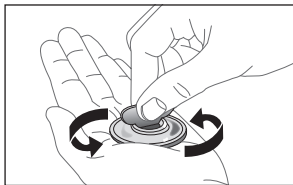
Fig. 3

TAKING YOUR BLOOD PRESSURE READING

Before taking a reading, review the section under Features & Assembly of your unit and ensure every piece is properly assembled. Reminder, it is important to avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to taking a reading.

NOTE: For the purposes of these instructions, the individual taking the reading will be referred to as the CAREGIVER, the person receiving the reading will be referred to as the PATIENT.

Applying the Cuff and Arm Positioning - Patient

1. Apply the cuff as instructed earlier in this manual.
2. Rest your elbow on a table with your palm and arm facing upward. Elevate your hand so that the wrist is at the same level as your heart, Fig 1. Relax your left hand.

Fig. 1

Preparing to Take the Measurement - Caregiver

3. Position yourself in front of the patient, seated at the same level.
4. Place the stethoscope eartips in your ears and place the chestpiece of the stethoscope on the inside of the patients arm. The chestpiece should be partially under the cuff directly over the brachial artery.
5. Dual-head stethoscopes feature a 2-sided rotating chestpiece, Fig 2.

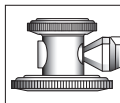
Fig. 2

Flat / larger side—general use

Bell shaped / smaller side—low-pitched sounds, typically used for detecting irregular heart sounds

Fig. 3

Tap the chestpiece; if you are unable to hear the tapping, rotate the chestpiece until an audible click is heard, ensuring the chestpiece is in the proper position.

6. Either hold the gauge in in your hand or position the gauge so it can be read without being moved, such as clipped to a thick notebook or folded towel in front of you.
7. Hold the inflation bulb with your free hand and turn the air release valve clockwise to close, Fig 3.

TAKING YOUR BLOOD PRESSURE READING

- Begin inflating the cuff slowly and steadily by pumping the inflation bulb until the gauge reads 30 mmHG ABOVE the last SYSTOLIC measurement. If you do not know the last SYSTOLIC measurement, inflate to approximately 180-200 mmHg. NEVER INFLATE THE CUFF ABOVE 280 mmHg.
- When you have reached the proper inflation level, slowly and steadily turn the air valve counter-clockwise at a rate of 2-3 mmHg per second. You will be able to watch the progress of the release on the gauge.
- As the pressure drops, listen closely for the pulse beat while watching the gauge. Note the first thumping sound you hear, this is the SYSTOLIC pressure, Fig 4.
- Allow the cuff to continue to deflate. You will hear changes in the sounds and then no sound at all. When you have heard the last sound, note the reading measurement, this is the DIASTOLIC pressure, Fig 5.
- Quickly turn the valve counter-clockwise to deflate the remaining air, Fig 6. Remove the cuff and record the measurement on the included chart.

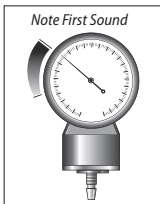
Fig. 4

Fig. 5

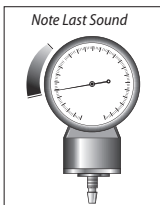
Fig. 6

IF YOU WISH TO TAKE ANOTHER MEASUREMENT, WAIT A MINIMUM OF 15 MINUTES BETWEEN READINGS TO ALLOW FOR THE BLOOD VESSELS TO RETURN TO NORMAL.

CARE & MAINTENANCE

Care & Maintenance

- The stethoscope may be cleaned with either soap and water or isopropyl alcohol. The eartpieces may be removed from the binaurals to facilitate cleaning. After each cleaning, all parts should be thoroughly rinsed and dried completely. Never immerse any part of the stethoscope in liquid. Never use solvents, oils or sterilization equipment to clean the unit.
- The chestpiece may be cleaned with the included cleaning cloth as needed. Soapy water or sopropyl alcohol may also be used.
- The diaphragm may also be removed and cleaned with soapy water or isopropyl alcohol. Rinse and dry completely after cleaning.
- The cuff may be spot cleaned with a mild-detergent as needed.
- Do not use any liquids on the gauge or cuff.
- Never attempt to iron or dry the cuff; if the cuff becomes wet, allow the cuff to air dry completely before use or storage.
- Do not attempt to remove the tube and bladder from the cuff as this may damage the cuff.
- Take care not to forcefully bend or kink the tubing during storage .
- It is recommended to clean the cuff every 200 uses.
- Avoid dropping or hitting the surface of the gauge or stethoscope chest piece.
- Only use included attachments and parts; do not use attachments from other brands or models as these may not be properly calibrated for use with this device and may result in measurement error.

Storage

- Avoid extreme temperatures.
- Do not store in direct sunlight, dust or excessive humidity.
- Storage conditions for device: Temperature 50°F–104°F (10°–40°C); 15%–85% RH.

When the cuff is deflated, make sure that the needle on the gauge is within the zero square range. If the needle is outside of the range, it needs to be recalibrate by a qualified technician. Consult with the included warranty card for sending your gauge in for calibration.

DEVICE & LABEL SYMBOLS

These symbols may appear on your device, instructions or packaging and may vary by make and model.

Symbol	Meaning
	Read This Manual—All included manuals should be read prior to first use. (Background color will be blue; reading symbol will be white)
	Warning—Symbol indicates a warning, prohibition or mandatory action that mitigates a risk that is not necessarily obvious to the device operator.
	Type BF Applied Parts—Indicates that a part of this unit comes in contact with the patient in order to carry out its intended function.
	Environment Protection—dispose of this product properly; consult with your local recycling ordinances for proper recycling and disposal.
	Keep Dry—This device should be kept dry; never submerge the unit or cuff. Consult with the Care and Maintenance section of this manual for information on cleaning your monitor.

SPHYGMOMANOMETER LIMITED WARRANTY

Congratulations on your purchase of a **Aneroid Sphygmomanometer**. Your Blood Pressure unit is covered by the following warranties commencing upon the date of purchase, and are subject to the following terms and conditions:

1. This Lifetime Limited Calibration Warranty is limited to the accuracy of the manometer gauge only. During the term of the warranty, Veridian Healthcare will repair or replace (at our option) any gauge found to be inaccurate due to normal use. This does not include gauges that have been misused or abused.
2. The two (2) year inflation system warranty covers all parts of the inflation system including inflation bulb, valve, inflation cuff and bladder. During the term of the warranty, Veridian Healthcare will repair or replace (at our option) any inflation system component(s) found to be defective in materials or workmanship.
3. If the manometer gauge, or any part of the inflation system fails, return the component that has failed prepaid to: Attn: Repair Department, Veridian Healthcare 1175 Lakeside Drive, Gurnee, Illinois 60031. The purchaser will be notified of any additional repairs required prior to completing the repair. Components will be subject to return shipping and handling charges, in addition to parts charges, if any, and repair charges not covered by this Limited Warranty.
4. This Limited Warranty constitutes Veridian Healthcare's only responsibility and obligation to repair and/or replace materials or components. We make no other express or implied warranties, arising by operation of law or otherwise, or any warranty of merchantability or fitness for a particular use or purpose whether or not the use of purpose has been disclosed to Veridian Healthcare in specifications, drawings or otherwise, and whether or not Veridian Healthcare's products are specifically designed and/or manufactured by Veridian Healthcare for the buyer's use or purposes, except for the limited warranty stated above.
5. Veridian will not be responsible for any indirect, incidental, special, consequential, or punitive damages or other loss, including, but not limited to damage to or loss of other property or equipment and personal injuries, whether to purchaser or others. Veridian shall in no event be liable to the purchaser for any amount in excess the cost or repair and/or replacement of the unit. Some states do not allow limitations on how long an implied warranty lasts or do not allow the exclusion or limitation of incidental or consequential damages, so the above limitations or exclusions may not apply to you.

EXCEPT AS PROVIDED FOR IN THIS LIMITED WARRANTY, ALL EXPRESS AND IMPLIED WARRANTIES AND CONDITIONS ARE DISCLAIMED, INCLUDING, WITHOUT LIMITATION, ANY IMPLIED WARRANTIES AND CONDITIONS OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.

STETHOSCOPE

LIFETIME LIMITED WARRANTY

Congratulations on your purchase of a **Stethoscope**. Your stethoscope is covered by the following warranty commencing upon the date of purchase, and are subject to the following terms and conditions:

1. This stethoscope lifetime limited warranty covers the stethoscope and component parts. During the term of this warranty, Veridian Healthcare will repair or replace (at our option) any stethoscope component(s) found to be defective in materials or workmanship.
2. If any part of the stethoscope fails, return the component that has failed prepaid to: **Attn: Repair Department, Veridian Healthcare 1175 Lakeside Drive, Gurnee, Illinois 60031**. The purchaser will be notified of any additional repairs required prior to completing the repair. Components will be subject to return shipping and handling charges, in addition to parts charges, if any, and repair charges not covered by this Limited Warranty.
3. This Limited Warranty constitutes Veridian Healthcare's only responsibility and obligation to repair and/or replace materials or components. We make no other express or implied warranties, arising by operation of law or otherwise, or any warranty of merchantability or fitness for a particular use or purpose whether or not the use of purpose has been disclosed to Veridian Healthcare in specifications, drawings or otherwise, and whether or not Veridian Healthcare's products are specifically designed and/or manufactured by Veridian Healthcare for the buyer's use or purposes, except for the limited warranty stated above.
4. Veridian will not be responsible for any indirect, incidental, special, consequential, or punitive damages or other loss, including, but not limited to damage to or loss of other property or equipment and personal injuries, whether to purchaser or others. Veridian shall in no event be liable to the purchaser for any amount in excess the cost or repair and/or replacement of the unit. Some states do not allow limitations on how long an implied warranty lasts or do not allow the exclusion or limitation of incidental or consequential damages, so the above limitations or exclusions may not apply to you.

EXCEPT AS PROVIDED FOR IN THIS LIMITED WARRANTY, ALL EXPRESS AND IMPLIED WARRANTIES AND CONDITIONS ARE DISCLAIMED, INCLUDING, WITHOUT LIMITATION, ANY IMPLIED WARRANTIES AND CONDITIONS OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.

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