612-0012-9850 ICE 4 3/4" x 6 1/2

ICE KOLD®

COMPRESA FRÍA INSTANTÁNEA

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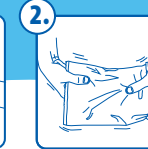
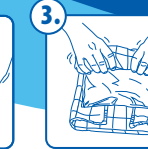
ICE KOLD®

INSTANT ICE COMPRESS

Disposable, single use only
Use only as directed, do not apply directly to skin
Best if activated between 32° and 80° F (0°-26.6° C)
For external use only

- 
Shake contents to the bottom of the bag. Locate inner pouch and squeeze firmly to rupture water pouch.
- 
Shake briefly to mix contents and ensure activation.
- 
Wrap compress in a soft cloth and apply to treatment area.
- 
Discard after single use.

CAUTION:

Do not reuse. Store in a dry place at room temperature. Do not refrigerate or freeze as this may result in frostbite. Always wrap compress with a soft cloth before application to treatment area; do not apply directly to skin. Do not use for more than 30 minutes; approximate temperature and therapeutic time: 33° F / 0.5° C, 20 minutes. For external use only. Properly dispose of after single use. Do not use bag if it is punctured, torn or leakage occurs.

WARNING:

Use carefully. May cause serious burns. Do not use over sensitive skin areas or in the presence of poor circulation. The unattended use of Ice Kold® Instant Ice Compress by children or incapacitated persons may be dangerous. Cold therapy is not recommended for people with circulatory problems, Raynauds Syndrome or diabetes without consent by your physician.

Harmful if swallowed. If swallowed, seek emergency medical attention immediately. If victim is drowsy or vomiting, do not give anything by mouth. If victim is conscious and alert and ingestion occurred within the last hour, induce vomiting.

If fluid comes in contact with eyes or open wound, flush with water and seek medical attention. If fluid comes in contact with skin, remove any contaminated clothing and wash the area with mild soap and water. Seek medical attention for any persistent symptoms including irritation or redness. Do not apply directly to skin.

Active Ingredients: Ammonium Nitrate, Water

DMi
DURO-MED INDUSTRIES
Lifestyle Products for Better Living


Distributed by:
Duro-Med Industries
Div. of DMS Holdings Inc.
1931 Norman Drive South
Waukegan, IL 60085 USA
www.mabisdmi.com
Made in China

612-0012-9850

0 4 1298 21212 6

#940-0612-0012 01/09
©2009 Duro-Med Industries

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pms 2915

pms 286