Dispatch® Hospital Cleaner Disinfectant Towels with Bleach

One-step cleaning, disinfecting and deodorizing of hard, nonporous surfaces

For more information,
contact your Clorox sales
representative or call
800-234-7700.

email:
healthcare@clorox.com
visit us:
www.cloroxhealthcare.com

- 1. REMOVE** gross soil if visible.* When disinfecting *C. difficile* spores, always clean surfaces prior to disinfecting.

- 2. WIPE** surface with towel until completely wet. Gloves should be worn.

- 3. To DISINFECT**, allow surface to remain wet for 1 minute. To kill TB, allow 2 minutes contact at room temperature. To kill *C. difficile* spores, allow 5 minutes contact.

- 4. WIPE** with a clean, damp cloth. Allow to air dry. Discard used towel.

This product may be used on stainless steel, glazed ceramic tile, hard plastics, nonporous vinyl, painted surfaces, Plexiglas®, laminated plastic countertops, fiberglass (sealed) surfaces, glazed porcelain and glass.

CAUTION: Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling. Do not use this product with ammonia or acids such as vinegar, rust removers or toilet bowl cleaners. Not intended for use on fabrics. Use by expiration date.

* This is a one-step product. Pre-cleaning is required only if blood, vomit, fecal matter or other soil is visible or in advance of disinfecting *C. difficile* spores. Use as directed on pre-cleaned hard, nonporous surfaces.

Plexiglas is a trademark of Arkema France Corporation.

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Plexiglas es una marca registrada de Arkema France Corporation.